

FIRST CRUSH™

AUTOMATICALLY TURNS PILLS INTO POWDER™



A TOP & BOTTOM CUP ARE REQUIRED FOR EVERY CRUSH. CLEAN WITH DISINFECTING WIPE BEFORE USE.



- 1 Press **ON** button to turn on unit
(Green light should be ON)



- 2 Place 5-6 medium-sized pills into the bottom cup
(Dimples point down on bottom cup)



- 3 Place top cup on top of bottom cup
(Dimples point up on top cup)



- 4 Place both cups into crusher;
firmly close lid until locked.



- 5 Push either **STANDARD** or **EXTRA GRIND** button.
Use **EXTRA GRIND** for hard and large pills.



- 6 Push the **BLUE** button on the front of the unit
and lift the lid to **OPEN**. Remove both cups
and tap **TOP** cup to remove any residue.

- 7 Add food to the serving cup; mix and serve.

- 8 **CHARGE** the FIRST CRUSH unit for a minimum of two (2) hours upon receipt. If your unit needs to be recharged, the **RED** light will blink. The unit is fully charged when the **RED** light turns off.

ONE YEAR LIMITED WARRANTY

First Wave Products Group, LLC. ("First Wave") warrants that for a period of one year from the date of purchase, this product will be free from defects in material and workmanship. First Wave, at its option, will repair or replace this product or any component of the product found to be defective during the warranty period. Replacement will be made with a new or remanufacturer product or component. If the product is no longer available, replacement may be made with a similar product of equal or greater value. This is your exclusive warrant.

This warranty is valid for the original purchaser from the date of initial retail purchase and is not transferable. Proof of purchase is required to obtain warranty performance. First Wave dealers, service centers or others selling First Wave products do not have the right to alter, modify or any way change the terms and conditions of this warranty.

This warranty does not cover normal wear of parts or damage resulting from any of the following: negligent use or misuse of the product, use of improper voltage or current, use of any consumable cup other than CompleteRX cups from First Wave, use contrary to the operating instructions, disassembly, repair or alteration by anyone other than the First Wave or an authorized First Wave service center. Further, the warranty does not cover: Acts of God, such as fire, flood, hurricanes and tornadoes.



FIRST WAVE PRODUCTS GROUP, LLC
99 Medtech Drive, Suite 109, Batavia, NY 14020
1-877-784-0269 • www.firstwaveproducts.com

What are the limits on First Wave Liability?

First Wave shall not be liable for any incidental or consequential damages caused by the breach of any express, implied or statutory warrant or condition.

Except to the extent prohibited by applicable law, any implied warranty or condition of merchantability or fitness for a particular purpose is limited in duration to the duration of the above warranty.

First Wave disclaims all other warranties, conditions or representations, express, implied, statutory or otherwise.

First Wave shall not be liable for any damages of any kind resulting from the purchase, use or misuse of, or inability to use the product including incidental, special, consequential or similar damages or loss of profits, or for any breach of contract, fundamental or otherwise, or for any claim brought against purchaser by any other party.

Some provinces, states or jurisdictions do not allow the exclusion or limitation of incidental or consequential damages or limitations on how long an implied warranty lasts to the above limitations or exclusions may not apply to you.

This warranty gives you specific legal rights, and you may also have other rights that vary from province to province, state to state or jurisdiction to jurisdiction.

How to Obtain Warranty Service?

If you have any questions regarding this warranty or would like to obtain warranty service please call 877-784-0269 and a convenient service center address will be provided to you.